

# CREATININE LIQUID REAGENT ASSAY

Renal and  
Pancreatic  
Marker

Enzymatic creatinine assays have less interference than older Jaffe creatinine assays. A study from leading clinical journal shows that interference with Jaffe creatinine assays may lead to inaccuracies in estimated glomerular filtration rates that are clinically important, especially in children and neonates.<sup>1</sup>

Diazyme's Creatinine Liquid Reagent assay is intended for the in vitro quantitative determination of creatinine in serum and urine. The assay is cost effective and provides outstanding reagent stability combined with the added convenience of instrument specific packing for several major instrument families.

## **DIAZYME CREATININE LIQUID REAGENT ASSAY ADVANTAGES**

- Accurate measurement of creatinine with enzymatic method
- Reduced interferences and no cuvette staining as seen in Jaffe method
- Assay is traceable to NIST material (IDMS)
- Measuring range: Serum: 0.14-13.56 mg/dL  
Urine: 0.14-141.25 mg/dL
- Liquid stable reagent, calibrator and controls requires no reagent preparation
- Wide range of instrument parameters available for simplifying implementation

## **REGULATORY STATUS**

510(k) Cleared; EU:  


# CREATININE LIQUID REAGENT ASSAY

Dual Vial  
Liquid Stable

## ASSAY SPECIFICATIONS

<b>Method</b>	Enzymatic Assay
<b>Sample Type &amp; Volume</b>	<ul style="list-style-type: none"> <li>Serum</li> <li>Urine</li> </ul> Sample Volume 8 $\mu$ L
<b>Method Correlation</b>	Serum: N = 55 y-intercept = 0.0643 Slope = 0.9467 R <sup>2</sup> = 0.9981  Urine: N = 51 y-intercept = -0.0518 Slope = 1.0002 R <sup>2</sup> = 0.9968
<b>Linearity</b>	Serum: 0.14 - 13.56 mg/dL (12 - 1200 $\mu$ mol/L)  Urine: 0.14 - 141.25 mg/dL (12 - 12500 $\mu$ mol/L)
<b>LOD</b>	12 $\mu$ mol/L (0.14 mg/dL)
<b>Calibration Levels</b>	1-Point Calibration
<b>Traceability</b>	Standard traceable NIST's SRM 914a
<b>Reagent On-Board Stability</b>	Opened: 4 weeks when stored at 2-8°C

### Creatinine Assay Procedure\*

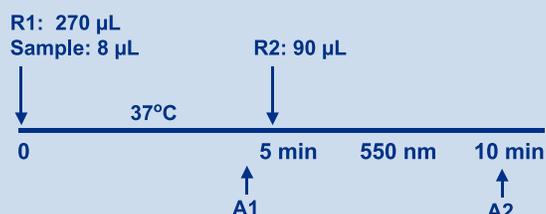

#### \*Analyzer Dependent

Parameter questions for Enzymatic Creatinine Assay should be addressed to Diazyme technical support. Please call 858.455.4768 or email [support@diazyme.com](mailto:support@diazyme.com)

## ASSAY PRECISION

The assay was evaluated according to Clinical Laboratory Standards Institute EP5-A guidelines. Four serum specimens were tested on a Hitachi 917 twice daily, in duplicates over 20 days.

Serum Testing	Within-Run Precision (80 Data Points)			
	0.75 mg/dL (66.3 $\mu$ M)	1.41 mg/dL (125 $\mu$ M)	4.11 mg/dL (346 $\mu$ M)	10.28 mg/dL (908.7 $\mu$ M)
<b>Mean mg/dL (<math>\mu</math>M)</b>	0.74 (65.4)	1.38 (122.3)	4.04 (357.5)	10.28 (908.7)
<b>SD mg/dL (<math>\mu</math>M)</b>	0.015 (1.3)	0.015 (1.37)	0.029 (2.54)	0.015 (1.3)
<b>CV%</b>	2.1%	1.1%	0.7%	0.1%
Serum Testing	Total Precision (80 Data Points)			
	0.75 mg/dL (66.3 $\mu$ M)	1.41 mg/dL (125 $\mu$ M)	4.11 mg/dL (346 $\mu$ M)	10.28 mg/dL (908.7 $\mu$ M)
<b>Mean mg/dL (<math>\mu</math>M)</b>	0.74 (65.4)	1.38 (122.3)	4.04 (357.5)	10.28 (908.7)
<b>SD mg/dL (<math>\mu</math>M)</b>	0.022 (1.9)	0.026 (2.29)	0.058 (5.11)	0.014 (12.4)
<b>CV%</b>	3.0%	1.9%	1.4%	1.4%

The assay precision was evaluated with urine samples with a modified EP10 protocol. Within-run precision; 21 replicates of commercial urine controls were tested. Total precision; 2 runs of each commercial urine control were performed consecutively for 5 days.

Urine Testing	Within-Run Precision (21 Data Points)		
	Level 1	Level 2	Level 3
<b>Mean mg/dL (<math>\mu</math>M)</b>	29.09 (2572)	87.1 (7711)	196.7 (17407)
<b>SD mg/dL (<math>\mu</math>M)</b>	0.1 (8.84)	0.27 (23.60)	0.90 (79.71)
<b>CV%</b>	0.36%	0.31%	0.46%
Urine Testing	Total Precision (20 Data Points)		
	Level 1	Level 2	Level 3
<b>Mean mg/dL (<math>\mu</math>M)</b>	29.86 (2640)	87.7 (7765)	195 (17265)
<b>SD mg/dL (<math>\mu</math>M)</b>	0.79 (69.8)	0.67 (59.2)	1.19 (105.2)
<b>CV%</b>	2.64%	0.76%	0.60%

## ASSAY INTERFERENCE

Interference for the Diazyme Creatinine Assay was evaluated on the Hitachi 917. The following substances normally present in serum produced less than 10% deviation at the listed concentrations:

Triglyceride:	up to 1000 mg/dL	Ascorbic Acid:	up to 10 mM
Bilirubin (Conjugate):	up to 30 mg/dL	Bilirubin:	up to 40 mg/dL
Hemoglobin:	up to 500 mg/dL		

The following substances normally present in urine produced less than 10% deviation at the listed concentrations:

Triglyceride:	up to 1000 mg/dL	Ascorbic Acid:	up to 10 mM
Bilirubin (Conjugate):	up to 40 mg/dL	Bilirubin:	up to 40 mg/dL
Hemoglobin:	up to 1000 mg/dL		

## DIAZYME LABORATORIES, INC.

12889 Gregg Court, Poway, CA 92064 USA  
 PO Box 85608, San Diego, CA 92186 USA  
 Tel: +1-858-455-4768 +1-888-DIAZYME  
[www.diazyme.com](http://www.diazyme.com) [sales@diazyme.com](mailto:sales@diazyme.com)

## DIAZYME EUROPE GMBH

Zum Windkanal 21, 01109 Dresden, Germany  
 Tel: +49-351-886-3300 Fax: +49-351-886-3366  
[sales@diazyme.de](mailto:sales@diazyme.de)

## SHANGHAI DIAZYME CO., LTD.

Building 8, 3879 Dongchuan Road  
 Minhang District, Shanghai, 200245  
 Tel: +86-21-54843802 Fax: +86-21-51320663  
[www.lanyuanbio.com](http://www.lanyuanbio.com) [service@lanyuanbio.com](mailto:service@lanyuanbio.com)

